# Angelcare®

Model AC300

*Movement Monitor*

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
## TABLE OF CONTENTS

4 Easy Steps to Operate Your Angelcare® Monitor .....	3
Warnings .....	4
Instructions .....	4
Testing Your Angelcare® Monitor .....	4
Movement Sensitivity Adjustment .....	4
Troubleshooting .....	5
Warranty .....	5
Distributors .....	9

### IMPORTANT NOTICE

#### USE OF THE ANGELCARE MONITOR WITH 2 SENSOR PADS

Connect the two sensor pads using the shorter wires. Position sensor pads as illustrated below. If you are using a bassinette or a small crib, position the one sensor pad with 2 cords in the middle of the crib under the mattress. Connect the long sensor pad cord to the nursery unit. The printed side of sensor pads **MUST** be face up. **DO NOT** tuck thick bedding under the edges of the mattress.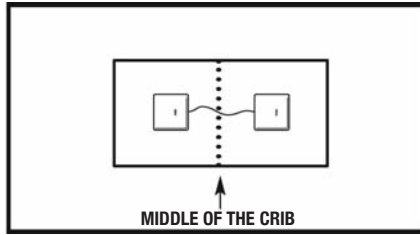

**STRANGULATION HAZARD.** Keep the adapter cord and the sensor pad cord out of the reach of children. Because of the serious strangulation risk to infants and toddlers, parents and caregivers **should never place the nursery unit within 3 feet (1 metre) of a crib.**

**A** Correct

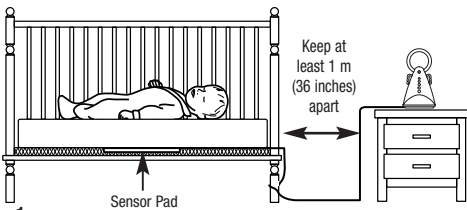
Figure 1

**B** Incorrect

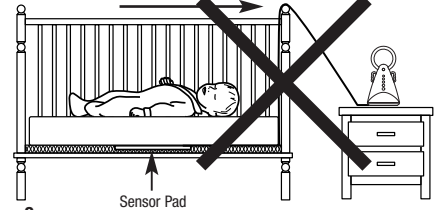
Figure 2

Figure 3



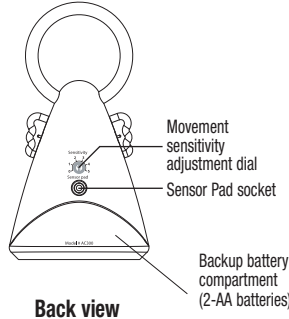
Figure 4
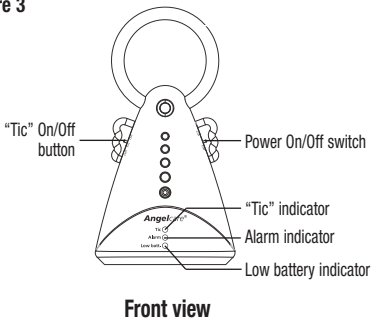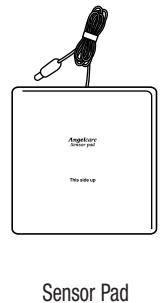
## 4 EASY STEPS TO OPERATE YOUR ANGELCARE® MONITOR

### STEP 1. SENSOR PAD

Position Sensor Pad as illustrated (Figures 1 and 2). The printed side of Sensor Pad **MUST** face up. **DO NOT** tuck thick bedding under the edges of the mattress.

**Sensor Pad must rest on a completely flat and rigid surface. If your crib has a flexible or uneven base (metal spring base, etc.), place a piece of 6 mm (1/4") plywood to cover the full size of the crib base.**

### STEP 2 MONITOR

- Insert 2 AA batteries into Monitor.
- Connect the Sensor Pad cord to the monitor (Figure 3).
- On the side of the Monitor, select 1 of the 2 functions of how you want to monitor your baby:
  - “Tic” On: To see green movement indicator light flash **AND** hear a “Tic” sound with every movement baby makes.
  - “Tic” Off: Same as above **BUT** without the “Tic” sound.

**NOTE:** The sensitivity adjustment has been preset. **DO NOT ADJUST UNLESS NECESSARY.** If poor capture of movement (red flashing light) or false alarm occurs, consult the Movement Sensitivity Adjustment section.

### STEP 3 TO USE MONITOR

Place baby in crib and switch Monitor to the ON position. When removing baby from crib, switch Monitor to the OFF position. If you forget to switch OFF, a “pre-alarm Tic” will sound after 15 seconds as a reminder.

### STEP 4 BATTERY MODE

The Angelcare® Movement Monitor is battery operated. Batteries will last approximately 3 to 4 months. When monitor produces a rapid clicking sound, change batteries promptly.

#### NOTE:

- Test your Angelcare® Monitor **BEFORE** first-time use and then on a periodic basis.
- **When monitor is in use, DO NOT use crib mobile or other crib accessories that produce vibrations. Alarm will not sound as monitor is still detecting movement.**
- **The Angelcare® Monitor is perfectly safe. The Sensor Pad and the cord are entirely passive. They carry no electrical current and do not radiate any form of energy.**
- Your monitor can be used in any crib or sleeping area (that has **NO** motion or vibration) designed to safely hold a baby.

## Warnings

- This product is **NOT** a medical device nor a device to prevent cases of Sudden Infant Death Syndrome (SIDS).
- This product is **NOT** a substitute for direct supervision of your baby. Check your baby's activities at regular intervals. Monitoring of premature babies, or those considered to be at risk, should be carried out only under the supervision of a doctor or health professional.
- The Sensor Pad may detect movement from many sources from **BOTH** inside and outside your baby's room, for example: fans, washing machines, loud music, etc. **Ensure that all vibration sources are eliminated before using the Angelcare® Monitor. Avoid contact with crib when monitor is in use. Refer to the Movement Sensitivity Adjustment and Troubleshooting sections.**
- **ALWAYS** place Monitor on flat surface in an upright position, away from baby's reach.
- This product should be serviced by qualified personnel. There are no user serviceable parts inside monitor. Refer all service questions to your distributor (see page 10).

## Instructions

**RETAIN INSTRUCTIONS:** Keep user's manual for future reference.

**WATER, MOISTURE AND HEAT DAMAGE:** Keep all components away from water (sink, bath tub, swimming pool) and places that emit heat (stove, radiator).

**VENTILATION: ALWAYS** place Monitor on flat surface in an upright position so that air can flow freely around it.

**CORD PROTECTION:** Avoid damage to the Sensor Pad cord. The cord is long enough to route so that it is not likely to be walked on or pinched by items placed upon or against it.

**CLEANING:** Disconnect Monitor before cleaning. **DO NOT** immerse in water. Use a dry cloth to keep Monitor free from dust. **DO NOT** spray with cleaning products or solvents. Wipe Sensor Pad with a very slightly damp cloth with an antiseptic or mild detergent.

**OBJECT AND LIQUID ENTRY:** Care should be taken so that objects do not fall onto Monitor and that liquids are not spilled into either Monitor or Sensor Pad through its openings.

**NON-USE PERIODS:** When not using monitor for long periods, remove the batteries from Monitor.

## Testing Your Angelcare® Monitor

**NOTE:** Test your Angelcare® Monitor BEFORE first-time use and then on a periodic basis.

The Angelcare® Monitor can be used outside of the crib, in a different sleeping area. Test monitor when moving Sensor Pad to a new location to ensure sensitivity is adjusted correctly to the new padding or mattress. Consult the Movement Sensitivity Adjustment section.

**Step 1** Switch the power and “Tic” function to the ON positions. Gently place your hand on the mattress. The green light will continue to flash with your every movement. Remove hand from mattress. Due to no movement, a “pre-alarm Tic” sound will emit after 15 seconds. 5 seconds later, the alarm will sound and the red light will flash continuously.


**NOTE:** If alarm does not sound, the Sensor Pad may be detecting a continuous vibration from the floor, a strong draft or the crib is being touched. Avoid contact with crib when monitor is in use. Place crib near a solid supporting wall and away from strong air currents. The sensitivity of the Sensor Pad may need to be reduced, consult the Movement Sensitivity Adjustment section.

**Step 2** To stop alarm, place your hand lightly on mattress for Sensor Pad to redetect movement. The red Alarm indicator light on the Monitor will continue to flash 1 minute (Figure 3).

**Step 3** When Monitor produces a rapid “clicking” sound, replace batteries promptly.

## Movement Sensitivity Adjustment

The Sensitivity Adjustment Dial has been preset to suit most situations. However in certain circumstances, the sensitivity may not be ideal. **DO NOT** increase sensitivity unless false alarms occurs and you have followed the advice in the Troubleshooting section for false alarms. Make a note of the pre-set position of the dial should you wish to return to it.

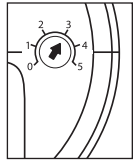The Alarm Indicator Light on the Monitor will flash red when Monitor barely detects movement. This could be a result of: baby being in a very deep sleep or baby having moved away from Sensor Pad to edge of crib. If alarm does **NOT** sound, no adjustment of sensitivity is required. If the Alarm Indicator Light flashes red most of the time with baby directly over the location of either Sensor Pad, a false alarm may occur. If this happens, adjust the sensitivity.

To adjust sensitivity, increase dial to the next setting. Retest monitor.

## TROUBLESHOOTING

Problem	Possible Cause	Solution
False alarms	<ul style="list-style-type: none"> <li>Baby was removed from crib and Monitor was not switched OFF .</li> <li>Sensor Pad cord not connected properly to Monitor.</li> <li>Sensor Pad is not in full contact with crib mattress, or crib mattress is not resting on a completely flat or rigid surface.</li> <li>Baby has moved away from Sensor Pad, to the edge of the crib.</li> </ul>	<ul style="list-style-type: none"> <li>Turn Monitor to the OFF position.</li> <li>Verify the connection between the Sensor Pad and Monitor. Unplug and reconnect the cord.</li> <li>Make sure there is no bedding between mattress and Sensor Pad. Sensor Pad must rest on a completely flat and rigid surface. If your crib has a flexible or uneven base (metal spring base, etc.), place a piece of 6 mm (1/4") plywood to cover the full size of the crib base.</li> <li>Make sure Sensor Pad is positioned as described in the 4 Easy Steps section. Sensitivity may need to be adjusted, consult the Movement Sensitivity Adjustment section.</li> </ul>
Alarm will not sound	<ul style="list-style-type: none"> <li>Monitor is detecting movement when crib is being touched.</li> <li>Monitor is detecting movement from outside the crib such as motorized devices, strong drafts, etc.</li> <li>Sensitivity is set too high.</li> </ul>	<ul style="list-style-type: none"> <li>Avoid contact with the crib when monitor is ON.</li> <li>Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside crib.</li> <li>You may need to decrease the sensitivity, consult the Movement Sensitivity Adjustment section.</li> </ul>
Alarm indicator light is flashing red (monitor is barely detecting movement)	<ul style="list-style-type: none"> <li>Sensor Pad is not in full contact with crib mattress, or crib mattress is not resting on a completely flat or rigid surface.</li> <li>Baby is in a very deep sleep or has moved away from Sensor Pad, to the edge of the crib.</li> </ul>	<ul style="list-style-type: none"> <li>Make sure there is no bedding between mattress and Sensor Pad. Sensor Pad must rest on a completely flat and rigid surface: place a piece of 6 mm (1/4") plywood to cover the full size of the crib base.</li> <li>Make sure Sensor Pad is positioned as described in the 4 Easy Steps section. Sensitivity may need to be adjusted, consult the Movement Sensitivity Adjustment section.</li> </ul>
No power	<ul style="list-style-type: none"> <li>Batteries are low or improperly installed.</li> <li>The Monitor is not ON.</li> </ul>	<ul style="list-style-type: none"> <li>Check / replace batteries.</li> <li>Make sure Monitor is ON.</li> </ul>

## LIMITED WARRANTY

---

Angelcare® warrants to the original owner of this product that the Angelcare® Movement Monitor (Model AC300) is free from defects in materials and workmanship for a period of one (1) year from the date of the original purchase. If this monitoring system fails to function properly when used as directed under normal conditions, within the one year period, Angelcare® will either repair or replace the product at our option free of charge, when it is returned complete and in its original packaging, prepaid postage, and insured to your distributor (see page 10).

The product must be accompanied by a proof of purchase, either a bill of sale, receipted invoice, or other proof that the monitoring system is within the warranty period. Angelcare® will bear the cost of repairing or replacing the product and shipping it back to you.

This warranty does not apply to a product that has been damaged as a result of improper maintenance, accidents, improper voltage supply or other misuse. The warranty is also void if the owner repairs or modifies the product in any way. Angelcare® is not liable for any incidental or consequential damages with regard to this product. The warranty also excludes any liability other than what is stated above. No other warranties are given.

LEGAL RIGHTS VARY FROM COUNTRY TO COUNTRY. SOME COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

WE ARE NOT IN ANY WAY RESPONSIBLE FOR ANY OUTCOME RESULTING FROM USE.

**IMPORTANT! This Angelcare® Monitor is a personal care product. Please do not return this product to the store. If you have questions or need help, please call your distributor (see page 10) before returning this product to the retailer.**

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Dorel UK Ltd
Unisar Inc.
\* Representing Unisar Inc.